

[Print](#) [Close](#)

## OPSITE<sup>◇</sup> FLEXIFIX<sup>◇</sup>

### Transparent Film Roll

[Overview](#)[Instructions for use](#)

#### Step 1

Cut off the desired length of film from roll.

#### Step 2

Fold the strip along its length to crack the protective papers. Remove the backing paper marked No.1 "remove first".

#### Step 3

Apply the adhesive film to desired area and smooth down.

#### Step 4

Remove the green squared grid carrier, marked No 3.

to the skin and lift off.

To remove the film, gently stretch the film parallel

**2010 SMITH & NEPHEW**

---