

[Print](#) [Close](#)

PROFORE◇

Multilayer High Compression Bandaging System

[Overview](#)[Instructions for
Use](#)

Step 1

Wash and dry leg and apply a moisturizing cream to the unbroken skin.

Apply PROFORE[®] in the following sequence:

Step 2

PROFORE[®] WCL sterile wound contact layer

Remove from the pack with sterile forceps and apply to the wound. Hold in place until covered by PROFORE[®] #1.

Step 3

PROFORE[®] #1 natural padding bandage

Apply from the base of the toes to the knee using a simple spiral technique and 50% overlap. Do not apply tension.

Step 4

Apply from the base of the toes to the knee, over PROFORE[®] #1, using a simple spiral technique and 50% overlap. Use tape to secure.

2010 SMITH & NEPHEW
