

680 Fora GD20 Glucose Monitoring System Instructions

Directions for Use:

1. Insert a test strip to turn on the meter.
2. Take an auto-disabling lancing device. Hold the lancing device firmly against the puncture site and puncture to get a drop of blood.
3. Gently squeeze around the punctured area to obtain enough blood.
4. While a blood drop is flashing on the display, hold and touch the absorbent hole of the test strip to the drop of blood until the confirmation window is completely covered.
5. After the meter counts to zero, the test result will appear along with date and time and will be automatically stored in the memory.
6. To turn off the meter, remove the test strip.
7. Discard used test strip and lancet device carefully.
8. Disinfect the meter as instructed in the Owner's Manual and change gloves before testing another patient.



Your blood glucose result will display the following:

Lo: Glucose level is less than 20 mg/dL (1.1 mmol/L)

Hi: Glucose level is greater than 600 mg/dL (33.3 mmol/L)

KETONE? : Glucose level is less than or equal to 240 mg/dL (13.3 mmol/L)