

473-04

XLARGE - REGULAR THIGH

KNIT IN USA OF US
AND IMPORTED YARNS
ASSEMBLED IN MEXICO



C.A.R.E.[®]
Control and Resist Embolisms

1 Pair

albahealth[®]

Graduated Compression Anti-Embolism Stockings

- **Color Coded Sizing**
Easy size identification
- **Top Inspection Port**
Provides easy access to toes without removing the stocking
- **Alignment Indicator**
Aids in proper positioning



APPLICATION INSTRUCTIONS

These stockings are designed for easy application, comfort and coolness while wearing. Please refer to the illustrations below.



STEP 1

Gather length of stocking down to heel pouch.

STEP 2

Stretch stocking open and fit over foot and heel.

STEP 3

Draw stocking evenly over ankle, calf and thigh to full length.
DO NOT TURN DOWN TOP OF STOCKING

STEP 4

Smooth out excess material with palm of hands and eliminate wrinkles in foot area by grasping toe section and pulling forward.


INDICATIONS: The use of Anti-Embolism Stockings is indicated in situations where there is likely to be impaired venous return from the lower extremities as in: Surgical patients, Pregnancy, Varicose veins, Geriatric/bedfast patients, Chronic venous insufficiency, Edema secondary to cast removal, Patients recovering from phlebitis.

CONTRAINDICATIONS: Anti-Embolism Stockings are not recommended for patients with: severe arteriosclerosis or other severe ischemic vascular diseases, including absence of pedal pulses; local skin disorders such as dermatitis, open ulcerations, infections, recent vein ligation, skin graft or gangrene in which stockings would interfere; massive peripheral edema associated with pulmonary edema; and severely deformed extremity.

CAUTION: Federal law restricts this device to sale by or on the order of a physician.

C.A.R.E.®

Control and Resist Embolisms Anti-Embolism Stocking Sizing Guide

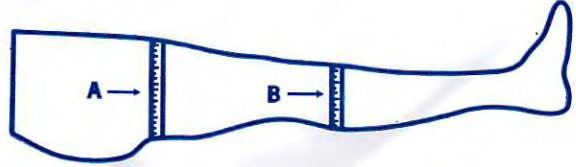

	Cat. No	Size	Calf Measures	Length Measures	Top	Toe Port		
					Color	Color		
KNEE	Regular	453-01	Small	up to 12"	up to 16"	White	Yellow	
		453-02	Medium	12" to 15"	16" to 17"	White	White	
		453-03	Large	15" to 17"	17" to 18"	White	Blue	
		453-04	X-Large	17" to 19"	18" to 19"	White	Green	
	Long	458-01	Small	up to 12"	up to 18"	Blue	Yellow	
		458-02	Medium	12" to 15"	18" to 19"	Blue	White	
		458-03	Large	15" to 17"	19" to 20"	Blue	Blue	
		458-04	X-Large	17" to 19"	20" to 21"	Blue	Green	
THIGH	Short	463-01	Small	up to 12"	up to 29"	Yellow	Yellow	
		468-01	Medium	12" to 15"	up to 29"	Yellow	White	
	Regular	473-01	Large	15" to 17"	up to 29"	Yellow	Blue	
		463-02	Small	up to 12"	29" to 33"	White	Yellow	
		468-02	Medium	12" to 15"	29" to 33"	White	White	
		473-02	Large	15" to 17"	29" to 33"	White	Blue	
		473-04	X-Large	17" to 19"	29" to 33"	White	Green	
		Long	463-03	Small	up to 12"	33" to 36"	Blue	Yellow
			468-03	Medium	12" to 15"	33" to 36"	Blue	White
			473-03	Large	15" to 17"	33" to 36"	Blue	Blue
473-05	X-Large		17" to 19"	33" to 36"	Blue	Green		

HOW TO MEASURE FOR PROPER FIT

FOR PROPER FIT - KNEE LENGTH

Size Based On

Circumference of "B"

Length Based On

Distance From Bend of Knee to
Bottom of Heel

FOR PROPER FIT - THIGH LENGTH

Size Based On

Circumference of "B"

Length Based On

Distance From "A" to Bottom of Heel

LAUNDERING GUIDELINES

- Machine Wash Lukewarm Water
- Air Dry/Tumble Dry Low Setting
- Do Not Iron

SINGLE PATIENT USE



CAUTION: If the circumference of the upper calf exceeds recommended measurements consult physician.

PRODUCT CONTENT

KNEE LENGTH 75% Nylon/13% Spandex/12% Polyester

THIGH LENGTH 86% Nylon/13% Spandex/1% Other Fiber

albahealth®

A Division of Encompass Group LLC

425 North Gateway Ave.

Rockwood, TN 37854

800 • 262 • 2404 www.encompassgroup.net

Bag #015404

Rev. 03/08-0

453-03

LARGE - REGULAR KNEE

KNIT IN USA OF US
AND IMPORTED YARNS
FINISHED IN MEXICO

20 - 30 mmHg



(01)10073614707112(10)213145

1 Pair

C.A.R.E.[®]

Control and Resist Embolisms

albahealth[®]

Graduated Compression Anti-Embolism Stockings

- **Color Coded Sizing**
Easy size identification
- **Top Inspection Port**
Provides easy access to toes without removing the stocking
- **Alignment Indicator**
Aids in proper positioning



APPLICATION INSTRUCTIONS

These stockings are designed for easy application, comfort and coolness while wearing. Please refer to the illustrations below.



STEP 1

Gather length of stocking down to heel pouch.

STEP 2

Stretch stocking open and fit over foot and heel.

STEP 3

Draw stocking evenly over ankle, calf and thigh to full length.

DO NOT TURN DOWN TOP OF STOCKING

STEP 4

Smooth-out excess material with palm of hands and eliminate wrinkles in foot area by grasping toe section and pulling forward.


INDICATIONS: The use of Anti-Embolism Stockings is indicated in situations where there is likely to be impaired venous return from the lower extremities as in: Surgical patients, Pregnancy, Varicose veins, Geriatric/bedfast patients, Chronic venous insufficiency, Edema secondary to cast removal, Patients recovering from phlebitis.

CONTRAINDICATIONS: Anti-Embolism Stockings are not recommended for patients with: severe arteriosclerosis or other severe ischemic vascular diseases, including absence of pedal pulses; local skin disorders such as dermatitis, open ulcerations, infections, recent vein ligation, skin graft or gangrene in which stockings would interfere; massive peripheral edema associated with pulmonary edema; and severely deformed extremity.

CAUTION: Federal law restricts this device to sale by or on the order of a physician.

C.A.R.E.[®]

Control and Resist Embolisms
Anti-Embolism Stocking
Sizing Guide

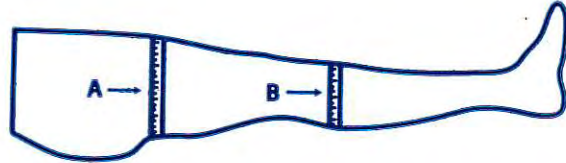

	Cat. No	Size	Calf Measures	Length Measures	Top Color	Toe Port Color	
KNEE	Regular	453-01	Small	up to 12"	up to 16"	White	Yellow
		453-02	Medium	12" to 15"	16" to 17"	White	White
		453-03	Large	15" to 17"	17" to 18"	White	Blue
		453-04	X-Large	17" to 19"	18" to 19"	White	Green
Long	458-01	Small	up to 12"	up to 18"	Blue	Yellow	
	458-02	Medium	12" to 15"	18" to 19"	Blue	White	
	458-03	Large	15" to 17"	19" to 20"	Blue	Blue	
	458-04	X-Large	17" to 19"	20" to 21"	Blue	Green	
Short	463-01	Small	up to 12"	up to 29"	Yellow	Yellow	
	468-01	Medium	12" to 15"	up to 29"	Yellow	White	
	473-01	Large	15" to 17"	up to 29"	Yellow	Blue	
THIGH	Regular	463-02	Small	up to 12"	29" to 33"	White	Yellow
		468-02	Medium	12" to 15"	29" to 33"	White	White
		473-02	Large	15" to 17"	29" to 33"	White	Blue
		473-04	X-Large	17" to 19"	29" to 33"	White	Green
Long	463-03	Small	up to 12"	33" to 36"	Blue	Yellow	
	468-03	Medium	12" to 15"	33" to 36"	Blue	White	
	473-03	Large	15" to 17"	33" to 36"	Blue	Blue	
	473-05	X-Large	17" to 19"	33" to 36"	Blue	Green	

HOW TO MEASURE FOR PROPER FIT

FOR PROPER FIT - KNEE LENGTH

Size Based On
Circumference of "B"

Length Based On

Distance From Bend of Knee to
Bottom of Heel

FOR PROPER FIT - THIGH LENGTH

Size Based On

Circumference of "B"

Length Based On

Distance From "A" to Bottom of Heel

LAUNDERING GUIDELINES

- Machine Wash Lukewarm Water
- Air Dry/Tumble Dry Low Setting
- Do Not Iron

SINGLE PATIENT USE



CAUTION: If the circumference of the upper calf exceeds recommended measurements consult physician.

PRODUCT CONTENT

KNEE LENGTH 75% Nylon/13% Spandex/12% Polyester

THIGH LENGTH 86% Nylon/13% Spandex/1% Other Fiber

albahealth

A Division of Encompass Group LLC

425 North Gateway Ave.

Rockwood, TN 37854

800 • 262 • 2404 www.encompassgroup.net

Bag #015404

Rev. 03/08-0