

473-04

XLARGE - REGULAR THIGH

KNIT IN USA OF US  
AND IMPORTED YARNS  
ASSEMBLED IN MEXICO



**C.A.R.E.**<sup>®</sup>  
Control and Resist Embolisms

1 Pair

albahealth<sup>®</sup>

**Graduated Compression Anti-Embolism Stockings**

- **Color Coded Sizing**  
Easy size identification
- **Top Inspection Port**  
Provides easy access to toes without removing the stocking
- **Alignment Indicator**  
Aids in proper positioning



**APPLICATION INSTRUCTIONS**

These stockings are designed for easy application, comfort and coolness while wearing. Please refer to the illustrations below.



**STEP 1**

Gather length of stocking down to heel pouch.

**STEP 2**

Stretch stocking open and fit over foot and heel.

**STEP 3**

Draw stocking evenly over ankle, calf and thigh to full length.  
**DO NOT TURN DOWN TOP OF STOCKING**

**STEP 4**

Smooth out excess material with palm of hands and eliminate wrinkles in foot area by grasping toe section and pulling forward.

**INDICATIONS:** The use of Anti-Embolism Stockings is indicated in situations where there is likely to be impaired venous return from the lower extremities as in: Surgical patients, Pregnancy, Varicose veins, Geriatric/bedfast patients, Chronic venous insufficiency, Edema secondary to cast removal, Patients recovering from phlebitis.


**CONTRAINDICATIONS:** Anti-Embolism Stockings are not recommended for patients with: severe arteriosclerosis or other severe ischemic vascular diseases, including absence of pedal pulses; local skin disorders such as dermatitis, open ulcerations, infections, recent vein ligation, skin graft or gangrene in which stockings would interfere; massive peripheral edema associated with pulmonary edema; and severely deformed extremity.

**CAUTION:** Federal law restricts this device to sale by or on the order of a physician.



**C.A.R.E.**<sup>®</sup>

**Control and Resist Embolisms  
Anti-Embolism Stocking  
Sizing Guide**

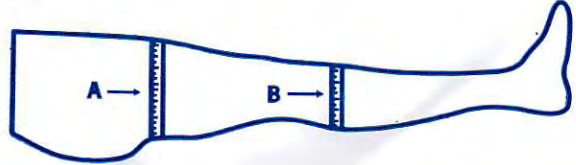

	Cat. No	Size	Calf Measures	Length Measures	Top	Toe Port	
					Color	Color	
<b>KNEE</b>	Regular	453-01	Small	up to 12"	up to 16"	White	Yellow
		453-02	Medium	12" to 15"	16" to 17"	White	White
		453-03	Large	15" to 17"	17" to 18"	White	Blue
		453-04	X-Large	17" to 19"	18" to 19"	White	Green
<b>KNEE</b>	Long	458-01	Small	up to 12"	up to 18"	Blue	Yellow
		458-02	Medium	12" to 15"	18" to 19"	Blue	White
		458-03	Large	15" to 17"	19" to 20"	Blue	Blue
		458-04	X-Large	17" to 19"	20" to 21"	Blue	Green
<b>THIGH</b>	Short	463-01	Small	up to 12"	up to 29"	Yellow	Yellow
		468-01	Medium	12" to 15"	up to 29"	Yellow	White
	473-01	Large	15" to 17"	up to 29"	Yellow	Blue	
	Regular	463-02	Small	up to 12"	29" to 33"	White	Yellow
		468-02	Medium	12" to 15"	29" to 33"	White	White
		473-02	Large	15" to 17"	29" to 33"	White	Blue
		473-04	X-Large	17" to 19"	29" to 33"	White	Green
	Long	463-03	Small	up to 12"	33" to 36"	Blue	Yellow
		468-03	Medium	12" to 15"	33" to 36"	Blue	White
		473-03	Large	15" to 17"	33" to 36"	Blue	Blue
473-05		X-Large	17" to 19"	33" to 36"	Blue	Green	

**HOW TO MEASURE  
FOR PROPER FIT**

**FOR PROPER FIT - KNEE LENGTH**

**Size Based On**

Circumference of "B"

**Length Based On**

Distance From Bend of Knee to Bottom of Heel

**FOR PROPER FIT - THIGH LENGTH**

**Size Based On**

Circumference of "B"

**Length Based On**

Distance From "A" to Bottom of Heel

**LAUNDERING GUIDELINES**

- Machine Wash Lukewarm Water
- Air Dry/Tumble Dry Low Setting
- Do Not Iron

**SINGLE PATIENT USE**



**CAUTION:** If the circumference of the upper calf exceeds recommended measurements consult physician.

**PRODUCT CONTENT**

**KNEE LENGTH** 75% Nylon/13% Spandex/12% Polyester

**THIGH LENGTH** 86% Nylon/13% Spandex/1% Other Fiber

**albahealth**<sup>®</sup>

A Division of Encompass Group LLC

425 North Gateway Ave.

Rockwood, TN 37854

800 • 262 • 2404 [www.encompassgroup.net](http://www.encompassgroup.net)

Bag #015404

Rev. 03/08-0



453-03

LARGE REGULAR KNEE

KNIT IN USA OF US  
AND IMPORTED YARNS  
FINISHED IN MEXICO

20 - 30 mmHg



1 Pair

**C.A.R.E.**<sup>®</sup>

Control and Resist Embolisms

albahealth<sup>®</sup>

**Graduated Compression Anti-Embolism Stockings**

- **Color Coded Sizing**  
Easy size identification
- **Top Inspection Port**  
Provides easy access to toes without removing the stocking
- **Alignment Indicator**  
Aids in proper positioning



**APPLICATION INSTRUCTIONS**

These stockings are designed for easy application, comfort and coolness while wearing. Please refer to the illustrations below.



**STEP 1**

Gather length of stocking down to heel pouch.

**STEP 2**

Stretch stocking open and fit over foot and heel.

**STEP 3**

Draw stocking evenly over ankle, calf and thigh to full length.

**DO NOT TURN DOWN TOP OF STOCKING**

**STEP 4**

Smooth-out excess material with palm of hands and eliminate wrinkles in foot area by grasping toe section and pulling forward.

**INDICATIONS:** The use of Anti-Embolism Stockings is indicated in situations where there is likely to be impaired venous return from the lower extremities as in: Surgical patients, Pregnancy, Varicose veins, Geriatric/bedfast patients, Chronic venous insufficiency, Edema secondary to cast removal, Patients recovering from phlebitis.


**CONTRAINDICATIONS:** Anti-Embolism Stockings are not recommended for patients with: severe arteriosclerosis or other severe ischemic vascular diseases, including absence of pedal pulses; local skin disorders such as dermatitis, open ulcerations, infections, recent vein ligation, skin graft or gangrene in which stockings would interfere; massive peripheral edema associated with pulmonary edema; and severely deformed extremity.

**CAUTION:** Federal law restricts this device to sale by or on the order of a physician.



# C.A.R.E.<sup>®</sup>

Control and Resist Embolisms  
Anti-Embolism Stocking  
Sizing Guide

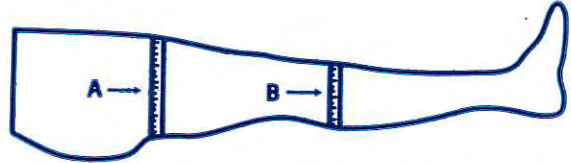

	Cat. No	Size	Calf Measures	Length Measures	Top Color	Toe Port Color	
<b>KNEE</b>	Regular	453-01	Small	up to 12"	up to 16"	White	Yellow
		453-02	Medium	12" to 15"	16" to 17"	White	White
		453-03	Large	15" to 17"	17" to 18"	White	Blue
		453-04	X-Large	17" to 19"	18" to 19"	White	Green
Long	458-01	Small	up to 12"	up to 18"	Blue	Yellow	
	458-02	Medium	12" to 15"	18" to 19"	Blue	White	
	458-03	Large	15" to 17"	19" to 20"	Blue	Blue	
	458-04	X-Large	17" to 19"	20" to 21"	Blue	Green	
Short	463-01	Small	up to 12"	up to 29"	Yellow	Yellow	
	468-01	Medium	12" to 15"	up to 29"	Yellow	White	
	473-01	Large	15" to 17"	up to 29"	Yellow	Blue	
<b>THIGH</b>	Regular	463-02	Small	up to 12"	29" to 33"	White	Yellow
		468-02	Medium	12" to 15"	29" to 33"	White	White
		473-02	Large	15" to 17"	29" to 33"	White	Blue
		473-04	X-Large	17" to 19"	29" to 33"	White	Green
Long	463-03	Small	up to 12"	33" to 36"	Blue	Yellow	
	468-03	Medium	12" to 15"	33" to 36"	Blue	White	
	473-03	Large	15" to 17"	33" to 36"	Blue	Blue	
	473-05	X-Large	17" to 19"	33" to 36"	Blue	Green	

## HOW TO MEASURE FOR PROPER FIT

### FOR PROPER FIT - KNEE LENGTH

Size Based On  
Circumference of "B"

Length Based On

Distance From Bend of Knee to  
Bottom of Heel

### FOR PROPER FIT - THIGH LENGTH

Size Based On

Circumference of "B"

Length Based On

Distance From "A" to Bottom of Heel

## LAUNDERING GUIDELINES

- Machine Wash Lukewarm Water
- Air Dry/Tumble Dry Low Setting
- Do Not Iron

## SINGLE PATIENT USE



**CAUTION:** If the circumference of the upper calf exceeds recommended measurements consult physician.

### PRODUCT CONTENT

**KNEE LENGTH** 75% Nylon/13% Spandex/12% Polyester

**THIGH LENGTH** 86% Nylon/13% Spandex/1% Other Fiber

**albahealth**

A Division of Encompass Group LLC

425 North Gateway Ave.

Rockwood, TN 37854

800 • 262 • 2404 [www.encompassgroup.net](http://www.encompassgroup.net)

Bag #015404

Rev. 03/08-0