

R_x Only

ZeniFoam Ag

Foam Dressing with Silver

No Adhesive / No Border

STERILE

Single Patient Use Only, Disposable

ZeniMedical



Part #'s: 10022-AG, 10045-AG, 10066-AG,
10077-AG, 10088-AG

DESCRIPTION

ZeniFoam Ag Foam Dressing with Silver(No Adhesive/ No Border) main includes polyurethane foam, polyvinyl alcohol, silver sulfate and silver chloride. It is a highly absorbent foam dressing with 0.35~0.40mg/cm³ ionic silver as the active component. The dressing provides an optimal moist wound healing environment, combining an effective antibacterial activity in the dressing with exudate management.

The dressing has board spectrum antibacterial effect for Gram positive and Gram negative bacteria. It has been shown to have 4 log bacterial reduction up to 7 days in vitro against the following bacteria: Staphylococcus aureus, Streptococcus pyogenes, Enterococcus faecalis, Escherichia coli, Klebsiella pneumoniae, and Pseudomonas aeruginosa.

ZeniFoam Ag Foam Dressing with Silver(No Adhesive/ No Border) is available in various sizes and shapes. The dressing is packaged in individual bag.

INDICATIONS FOR USE

ZeniFoam Ag Foam Dressing with Silver(No Adhesive/ No Border) is indicated for exudate absorption and the management of partial to full thickness wounds, including leg ulcers, pressure ulcers, diabetic foot ulcers, second-degree burns, donor sites, post-operative wounds and skin abrasions.

INSTRUCTIONS FOR USE

- 1.Clean the wound with sterile saline water and dry the surrounding skin gently.
- 2.Choose the right size of dressing to cover the wound according to the wound area.
- 3.Cover and secure the **ZeniFoam Ag** Foam Dressing with Silver(No Adhesive/ No Border) with a secondary dressing.
- 4.Dressing change frequency will depend on patient condition and the level of exudate, such as every 24 hours or longer, but could not be more than 7 days.
- 5.Gently remove the secondary dressing. If the wound appears dry, saturate the dressing with sterile saline water prior to removal.

CONTRAINDICATIONS

Do not use ZeniFoam Ag Foam Dressing with Silver(No Adhesive/ No Border) on patients with known sensitivity to silver.

WARNINGS

- 1.This product is single use. Do not use if pouch is damaged or open.
- 2.Do not suggest using this product on dry wound.
- 3.Frequent or prolonged use of this product may result in permanent discoloration of skin.
- 4.The silver is only present as a preservative within the dressing.
- 5.ZeniFoam Ag Foam Dressing with Silver(No Adhesive/ No Border) is intended for short-term use , should not be used more than 28 days over multiple dressing changes.
- 6.Reduction in colonization or microbial growth on the device has not been shown to correlate with a reduction in infections in patients. Clinical studies to evaluate reduction in infection have not been performed.
- 7.**ZeniFoam Ag** Foam Dressing with Silver(No Adhesive/ No Border) is not intended for use on third-degree burns.

ADVERSE REACTIONS

No adverse effects have been reported.

SPECIFICATIONS

Product No.	Size
10022-AG	2"×2"
10045-AG	4"×5"
10066-AG	6"×6"
10077-AG	7"×7"
10088-AG	8"×8"

Sterilization

The products are sterilized by irradiation, single use only.

Storage

Please store the product in sealed, cool and dry place avoiding light.

Shelf life

3 years from date of manufacture.

Manufactured for:

ZeniMedical
1410 East Weisgarber Road
Knoxville, Tennessee 37909

Rev. 10/2017