

R<sub>x</sub> Only

**ZeniFoam Ag GENTLE BORDER**  
Foam Dressing with Silver  
with Silicone Adhesive Border  
STERILE  
Single Patient Use Only, Disposable



**DESCRIPTION**

**ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border can absorb exudates, maintains a moist wound healing environment and has good antibacterial preservative properties. **ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border consists of 0.25~0.35mg/cm<sup>2</sup> silver ions, and it has been shown that antibacterial preservative effectiveness within the dressing for up to 7days, as demonstrated in vitro.

**ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border has two variants: Non-Border and Border, they have similar structure and materials.

**MATERIAL CONSISTS OF:**

- 1.A vapor permeable waterproof polyurethane top film.
- 2.A non-woven fabric layer in Border version.
- 3.An absorbent polyurethane foam pad containing polyvinyl alcohol, silver sulfate and silver chloride.
- 4.A soft silicone gel wound contact layer.
- 5.Release film of PE.

**INDICATIONS**

**ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border is indicated for the management of exuding wounds, such as leg and foot ulcers, pressure ulcers, traumatic and surgical wounds, superficial and partial thickness burns.

Silver compounds present in the dressing helps reduce bacterial colonization in the dressing.

**INSTRUCTIONS FOR USE**

- 1.Clean the wound with sterile saline water and dry the surrounding skin gently.
- 2.Choose the right size of dressing and remove the release films to cover the wound according to the wound area. The dressing should overlap the dry surrounding skin by at least 1~2 cm for the smaller sizes and 5cm for the larger sizes.
- 3.The frequency of replacing depends on patient's condition and the exudates situation, such as every 24 hours or longer, but cannot more than 7 days.

**CONTRAINDICATIONS**

Do not use **ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border on patients with a known sensitivity to silver or any other contents of the dressing.

**WARNINGS**

- 1.**ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border is single use. Do not reuse.
- 2.**ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border is sterile. Do not use it if the pouch is damaged or opened prior to use.
- 3.Do not suggest using this product on dry wound.
- 4.Do not use **ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border together with oxidant, such as hypochlorite solutions or hydrogen peroxide.
- 5.**ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border may cause transient discoloration of the wound bed and surrounding skin. Frequent or prolonged use of this product may result in permanent discoloration of skin.
- 6.**ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border does not replace the need for systemic therapy or other adequate infection treatment in the event of clinical infection.
- 7.Do not use **ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border during radiation treatment or examinations, such as X-ray, magnetic resonance imaging [MRI], ultrasound or diathermy. The **ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border is MR unsafe.
- 8.Avoid contact with electrodes or conductive gels during electronic measurements, such as electrocardiograms [ECG], and electroencephalograms [EEG].
- 9.Simpuritytm Silicone Ag Foam Dressing is only used for external use.
- 10.The silver is only present as a preservative within the dressing.
- 11.**ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border is intended for short-term use up to 28 days.
- 12.Reduction in colonization or microbial growth on the device has not been shown to correlate with a reduction in infections in patients. Clinical studies to evaluate reduction in infection have not been performed.
- 13.**ZeniFoam Ag GENTLE BORDER** Foam Dressing with Silver with Silicone Adhesive Border is not intended for use on third-degree burns.

**SPECIFICATIONS**

Code	Size
30044-AG	4"x4"
30066-AG	6"x6"
30077-AG	7"x7"

**Storage and disposal**

1. Please store the product in cool and dry place, and avoid light.
2. Disposal should be handled according to local environmental procedures.

**Shelf life:** 2 years from date of manufacture

**Manufactured for:**

ZeniMedical  
1410 East Weisgarber Road  
Knoxville, Tennessee 37909