

Adapt™

(US) (GB) Universal Remover Instructions

Alcohol free – No Sting! For external use only.

Removes tapes, barriers, cements and their adhesive residue from the skin.

Starting at the edge, gently peel back the skin barrier or adhesive as you continue to apply the Adapt universal remover. When the barrier or tape is completely removed from the skin, gently wipe off any remaining residue with Adapt universal remover. Thoroughly wipe the skin. Rinse well and dry. Store in a cool, dry place.

WARNING: Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, and vagina). Discontinue use if irritation occurs.

FLAMMABLE: No smoking. Do not use near flame or any incandescent material, electrical appliance or source of heat. Keep away from sources of ignition.

CONTENTS: HEXAMETHYLDISILOXANE