


How to use

Opening the box

Hold the turquoise tab and pull back to open. Do not tear the tab as it allows you to re-seal the box.

Elastic Barrier Strips

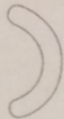

Stretch the strip slightly to release the protective paper backing. Peel back one piece of backing to expose the adhesive.




Place the strip where desired on the oval or round ostomy barrier. Be sure to position it around the outer edge of the barrier so the strip overlaps both the barrier and the surrounding skin.


Remove the remaining piece of paper backing and smooth the strip onto the skin. Repeat if necessary.

Elastic Barrier Strips - Y-shape

Application

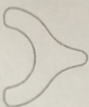

Stretch the strip slightly to release the protective paper backing. Peel back one piece of backing to expose the adhesive.

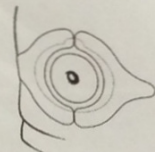


Place the strip where desired on the ostomy barrier, where you need additional support. Be sure to position it around the outer edge of the barrier so the strip overlaps both the barrier and the surrounding skin.


Remove the remaining piece of paper backing and smooth the strip onto the skin. Repeat if necessary.

Elastic Barrier Strips - Straight

Application

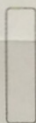

Stretch the strip slightly to release the protective paper backing. Peel back one piece of backing to expose the adhesive.

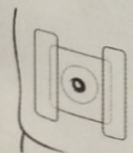

Place the strip where desired on the ostomy barrier, where you need additional support. Be sure to position it along the outer edge of the barrier so the strip overlaps both the barrier and the surrounding skin.

Remove the remaining piece of paper backing and smooth the strip onto the skin. Repeat if necessary.

Do not stretch the product when applying.

You can place a strip wherever you desire extra support.

Removal of the strips

When it's time to remove, the strips may come off together with your ostomy barrier. If needed, loosen the edges of the strips and gently stretch to remove.

Disposal of the strips

The product is intended for single use only and should be disposed of with normal household waste. Do not flush the product down the toilet.