
TOUCHLESS® PLUS

INTERMITTENT CATHETER SYSTEM



UROLOGY

BARD | MEDICAL

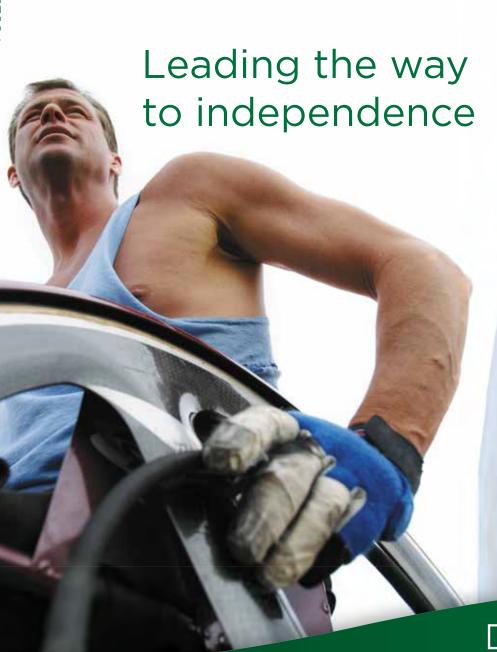

Bard works closely with leading urologists to develop innovative products that enhance patient safety, convenience and ease of use. We are committed to using superior materials, then engineering them for the comfort and reliability needed to lead a life of independence.

Bard has been a leader in urology since
1907, and remains the product of choice
for hospitals, rehabilitation facilities
and consumers. With a broad range of
products that set the standard for quality
and innovation, Bard is dedicated to
developing products that improve patient
care and reduce costs associated with
urinary tract infections.

Trust the choice of medical professionals.

Ease of Use

Easy to insert, easy to use, easy to remove - the features needed to enhance convenience and usage.

- Catheter cap has pull ring for easy removal especially for people with limited dexterity.
- Pre-lubricated catheter reduces discomfort during insertion.
- Large 1100cc collection bag is durable, virtually eliminating the risk of leaks.



The Bard® Touchless® Plus Intermittent Catheter System is a closed system, designed to help reduce the risk of urinary tract infections.

• Insertion tip positions the catheter in the urethra, bypassing the meatus to help reduce UTIs.

 Catheter guide allows control of catheter insertion with the touch of a finger.

 Tapered neck design decreases excoriation associated with sharp corners.



Sterile, Easy to Use:

- 1. Remove cap 1 from insertion tip using pull ring.
- 2. Slide the pre-lubricated catheter 2 halfway into the insertion tip 3 and insert into urethra.
- **3.** Holding the patented guide **4** with non-dominant hand, use dominant hand to grasp catheter through bag **5** and push catheter into urethra.
- **4.** Once bag is filled, withdraw catheter through insertion tip, snap on cap and dispose.

Complete instructions for use included in product packaging.

Your System, Your Choice

- Choice of red rubber or vinyl catheters based upon your preference and needs.
- Variety of catheter sizes, and available with or without accessories.

Product Number	Description	French Size	Latex-Free	HCPC Code
Accessories includ	led - 3 PVI swabs, 2	2 latex-free glo	oves, underpa	nd
4A5108	Vinyl	8 Fr.	Yes	A4353
4A5110	Vinyl	10 Fr.	Yes	A4353
4A5142	Vinyl	12 Fr.	Yes	A4353
4A5144	Vinyl	14 Fr.	Yes	A4353
AA5146	Vinyl	16 Fr.	Yes	A4353
4A5146 4A7114	Vinyl Coude	14 Fr.	Yes	A4353
Z 4A7116	Vinyl Coude	16 Fr.	Yes	A4353
4A5042	Red Rubber	12 Fr.	No	A4353
4A5044	Red Rubber	14 Fr.	No	A4353
4A7044	Red Rubber Coude	14 Fr.	No	A4353
Accessories not in	cluded (catheter ar	nd collection b	ag only)	
4A6110	Vinyl	10 Fr.	Yes	A4351
4A6142	Vinyl	12 Fr.	Yes	A4351
4A6144	Vinyl	14 Fr.	Yes	A4351
4A6146	Vinyl	16 Fr.	Yes	A4351
4A8114	Vinyl Coude	14 Fr.	Yes	A4352
4A8116	Vinyl Coude	16 Fr.	Yes	A4352
4A6044	Red Rubber	14 Fr.	No	A4351

NOTE: Reimbursement policies vary widely from insurer to insurer and reimbursement policies of the same insurer may vary in different sections of the U.S. As reimbursement policies are subject to change, the information contained herein may not be accurate at the time of use. Prior to submission of a claim for reimbursement, the user should contact the insurer (i.e., Medicare, Medicaid, or private payer) to verify applicable codes and reimbursement levels. C. R. Bard, Inc. does not guarantee that the purchase of any similar item is reimbursement in whole or in part, by any public or private health care insurer.

For more information, please contact us at 1-800-526-4455 or visit www.bardmedical.com.



C.R. Bard, Inc. Covington, GA 30014 Please consult product labels and inserts for any indications, contraindications, hazards, warnings, cautions and directions for use.

Bard and Touchless are trademarks and/or registered trademarks of C.R. Bard, Inc. ©2012 C.R. Bard, Inc. All Rights Reserved. Printed in U.S.A. 1203-83 R04/12 THP P07/12 5M