



HYDROGEL SHEET WOUND DRESSING

For the management of dry to lightly exuding wounds

DESCRIPTION

Aquaderm is a translucent flexible absorbent hydrogel sheet dressing for partial or full thickness wounds. AquaDerm absorbs exudate and maintains its shape while supporting a moist wound healing environment. AquaDerm leaves no residue and minimizes dressing change related pain.

INDICATIONS FOR USE

For the management of partial or full thickness wounds, including: pressure ulcers, minor burns, surgical wounds, skin abrasions and radiation tissue damage.

CONTRAINDICATIONS

AquaDerm should not be used on full thickness burns or on people who are sensitive or allergic to the dressing and its components.

DIRECTIONS (Please use under the guidance of a healthcare professional.)

PREPARE WOUND SITE AND SURROUNDING SKIN:

1. Cleanse wound with advised liquid to remove residue according to local infection control protocol. Deep wounds should be well irrigated.
2. Skin around wound should be clean and dry, use skin barrier prep as needed.

TO APPLY:

1. Remove dressing from package.
2. Dressing may be cut to size prior to application.
3. Remove backing paper from dressing.
4. Apply gel surface directly to wound, overlapping 1-2 inches of dry skin.
5. Cover with appropriate secondary dressing that maintains a moist wound bed.

AquaDermTM

HYDROGEL SHEET WOUND DRESSING

For the management of dry to lightly exuding wounds

CHANGING THE DRESSING:


- Dressing may be left in place for up to 7 days.
- Change the dressing if drainage shows through the back of the dressing, or if the edge of the dressing becomes loose; or if the condition of the wound or surrounding skin causes concern; or as directed by a qualified healthcare professional.
- Dressing may be moistened to ease removal.
- Carefully remove adhesive dressings by holding the edge of the dressing and slowly pulling it parallel to the skin.
- Dispose of in accordance with local guidance.

WARNINGS

- For single use only.
- If signs of infection are detected, a qualified healthcare professional should be contacted to decide on the next course of action.
- If signs of an allergic reaction are detected, discontinue use of the product and seek advice from a qualified healthcare professional.
- Product is sterile unless the package is opened or damaged. Do not resterilize.

OTHER INFORMATION

- Store in a dry place at room temperature.
- Keep away from sunlight.
- Not made with natural rubber latex.

DEPTH / STAGE	ABSORBENCY
PARTIAL FULL 1 2 3 4	 LOW
For Partial and Full thickness wounds / Pressure ulcers stages 2-4	

DermaRite[®]