

DermaFilm™ X-Thin Clear

CLEAR HYDROCOLLOID WOUND DRESSING WITH GRID

Protects wound and maintains a moist wound environment

DESCRIPTION

DermaFilm X-Thin Clear is a transparent hydrocolloid wound dressing for shallow non-infected wounds and shallow pressure ulcers. DermaFilm X-Thin Clear allows visualization of the wound, provides protection and normothermia, maintains a moist wound environment and supports autolytic debridement.

INDICATIONS FOR USE

May be used as a primary or secondary dressing. For the management of low exuding non-infected shallow wounds including partial thickness and shallow full thickness ulcers.

CONTRAINDICATIONS

DermaFilm should not be used on infected wounds or on people who are sensitive or allergic to the dressing and its components.

DIRECTIONS (Please use under the guidance of a healthcare professional.)

PREPARE WOUND SITE AND SURROUNDING SKIN:

1. Cleanse wound with advised liquid to remove residue according to local infection control protocol. Deep wounds should be well irrigated.
2. Skin around wound should be clean and dry, use skin barrier prep as needed.

TO APPLY:

1. Warm packaged dressing in hands prior to application.
2. Remove dressing from package.
3. Dressing may be cut to size prior to application.
4. Remove backing paper from dressing.
5. Apply sticky side of dressing to moist wound bed overlapping 1-2 inches of dry skin; gently smooth into place. Hold in place for several seconds to improve adhesion.

DermaFilm™ X-Thin Clear

CLEAR HYDROCOLLOID WOUND DRESSING
WITH GRID

Protects wound and maintains a moist wound environment

CHANGING THE DRESSING:


- Dressing may be left in place for up to 7 days.
- Change the dressing if drainage seeps through the back of the dressing, or if the edge of the dressing becomes loose; or if the condition of the wound or surrounding skin causes concern; or as directed by a qualified healthcare professional.
- Dressing may be moistened to ease removal.
- Carefully remove adhesive dressings by holding the edge of the dressing and slowly pulling it parallel to the skin.
- Dispose of in accordance with local guidance.

WARNINGS

- For single use only.
- If signs of infection are detected, a qualified healthcare professional should be contacted to decide on the next course of action.
- If signs of an allergic reaction are detected, discontinue use of the product and seek advice from a qualified healthcare professional.
- Product is sterile unless the package is opened or damaged. Do not resterilize.

OTHER INFORMATION

- Store in a dry place at room temperature.
- Keep away from sunlight.
- Not made with natural rubber latex.

DEPTH / STAGE	ABSORBENCY
PARTIAL FULL	
1 2 3 4	LOW
For Partial and Full thickness wounds / Pressure ulcers stages 2-4	

DermaRite®