

DermaView™

TRANSPARENT FILM WOUND DRESSING WITH LABEL AND GRID

For the management of dry to lightly exuding wounds

DESCRIPTION

DermaView is a non-absorbent, moisture vapor permeable, self-adhesive transparent film wound dressing for non-infected wounds with minimal drainage. Allows visualization of wound, maintains a moist wound bed and helps prevent bacterial contamination of wound.

INDICATIONS FOR USE

May be used as a primary or secondary dressing. For the management of dry to lightly exuding wounds, including: non-infected wounds, partial thickness ulcers, shallow full thickness ulcers.

CONTRAINDICATIONS

DermaView should not be used on people who are sensitive or allergic to the dressing and its components.

DIRECTIONS (Please use under the guidance of a healthcare professional.)

PREPARE WOUND SITE AND SURROUNDING SKIN:

1. Cleanse wound with advised liquid to remove residue according to local infection control protocol. Deep wounds should be well irrigated.
2. Skin around wound should be clean and dry, use skin barrier prep as needed.

TO APPLY:

1. Remove dressing from package.
2. If applicable, remove middle paper panel from the front (non-sticky) side of dressing.
3. Remove backing paper from dressing.
4. Apply sticky side of dressing to moist wound bed overlapping 1-2 inches of dry skin; gently smooth into place.
5. If applicable, remove paper frame from front (non-sticky) side of dressing.

DermaView™

TRANSPARENT FILM WOUND DRESSING WITH LABEL AND GRID

For the management of dry to lightly exuding wounds

CHANGING THE DRESSING:


- Dressing may be left in place for up to 3 days.
- Change the dressing if drainage seeps through the back of the dressing, or if the edge of the dressing becomes loose; or if the condition of the wound or surrounding skin causes concern; or as directed by a qualified healthcare professional.
- Dressing may be moistened to ease removal.
- Carefully remove adhesive dressings by holding the edge of the dressing and slowly pulling it parallel to the skin.
- Dispose of in accordance with local guidance.

WARNINGS

- For single use only.
- If signs of infection are detected, a qualified healthcare professional should be contacted to decide on the next course of action.
- If signs of an allergic reaction are detected, discontinue use of the product and seek advice from a qualified healthcare professional.
- Product is sterile unless the package is opened or damaged. Do not resterilize.

OTHER INFORMATION

- Store in a dry place at room temperature.
- Keep away from sunlight.
- Not made with natural rubber latex.

DEPTH / STAGE	ABSORBENCY
PARTIAL FULL 1 2 3 4	 NONE
For Partial and Full thickness wounds / Pressure ulcers stages 1-4	

DermaRite®