

**US GB Karaya Powder**

**For Peristomal Skin Protection:**

Skin should be clean and dry before applying the powder. Karaya is a natural, non-sterile product. Do not use if patient has a compromised immune system and/or may be at risk for infections unless directed by a healthcare professional.

**For external use only.**

**To Open:** Rotate cap counter clockwise. Squeeze bottle to dispense powder.

**To Use:** Wash skin with lukewarm water. Gently pat dry. Apply powder.

*On eroded or weeping skin,* Karaya Powder will combine with skin's moisture to form a paste. A temporary tingling sensation may occur.

*On intact skin,* moisten with water and apply powder.

Allow Karaya Powder to dry until a crust forms. Gently brush away any loose powder and apply pouch.

**CONTENTS:** GRADE #1 POWDERED GUM KARAYA