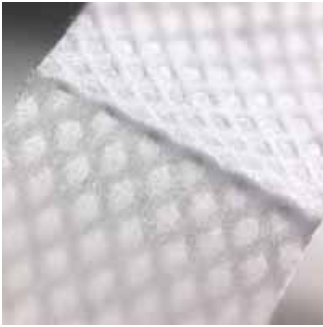


[Print](#) [Close](#)

SKIN-PREP◇

Protective Barrier Wipe

**Overview****Instructions for
Use**

For OTC applications, please follow Directions and Instructions for Use provided by your physician, nurse or healthcare professional. If no instructions were provided, follow the directions below:

Step 1

Skin should be clean and dry prior to application of SKIN-PREP[®].

Step 2

Apply a uniform coating over entire area you wish to protect, i.e., around stoma, under dressing, catheter site, etc.

Step 3

SKIN-PREP[®] will dry in approximately 30 seconds. If you miss an area that you intended to cover, wait until the original area of application has dried, then reapply to the missed area.

Step 4

For maximum protection, an optional second coating of SKIN-PREP[®] may be applied and allowed to dry before covering area with dressing or other adhesive product.

2010 SMITH & NEPHEW
