

# Adapt Lubricating Deodorant

Ostomy Care Tips

## General Information

- Adapt Lubricating Deodorant destroys odors, eases emptying and helps prevent pouch static and sticking.
- It is not a perfume or masking agent.
- Adapt Lubricating Deodorant should be used in drainable and closed pouching systems only.



Figure 1

## For Drainable Pouches

1. Apply pouch.
2. **Bottle:** add approximately 5ml or 1 tsp. (Figure 1).  
**Packet:** add the complete contents of the packet. (Figure 2)  
Close the pouch in the normal manner.
3. Rub to thoroughly coat the inside of the pouch (Figure 3).  
Repeat Steps 1 and 2 after every emptying.



Figure 2

## For Closed Pouches

1. Do not remove release paper on barrier to add product.
2. **Bottle:** add approximately 5ml or 1tsp. (Figure 4).  
**Packet:** add the complete contents of the packet. Apply your pouch.
3. Rub to thoroughly coat the inside of the pouch (Figure 3).

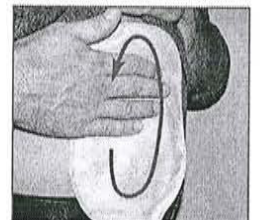


Figure 3

## Frequently Asked Questions

**Q: Does it matter if the Adapt Lubricating Deodorant comes in contact with the filter on the pouch?**

**A:** When coating the inside of the pouch, it is best to avoid the filter.

**Q: Is Adapt Lubricating Deodorant covered by Medicare?**

**A:** Yes, Adapt Lubricating Deodorant is billed per ounce using code A4394 (Ostomy deodorant for use in ostomy pouch, liquid). Medicare has not published a usual maximum quantity for this code. Both the 8 oz. bottle and 8 ml packets are billed using the same code. When billing the bottle, 8 units of A4394 will be billed. When using the packets (50 per box), 13 units of A4394 will be billed.

**Q: Do I need to use the Adapt Lubricating Deodorant every time I empty my pouch?**

**A:** Yes. For the product to work effectively, it must be reinserted each time your pouch is drained.

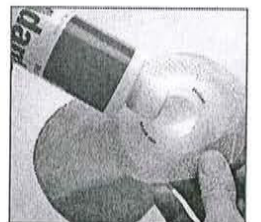


Figure 4



Attention to Detail. Attention to Life.